

Set-up Instructions-Arm Strap for the Flex-coil® Swivel System Page 1

U.S. Patent No. 5,163,923 • Canadian Patent No. 2,119,376



The blue arm strap is designed for comfort and convenience. It permits one person to carry or walk a patient, along with the fluids, the Flex-coil® Swivel Set, and I.V. tubing, while the patient continues to receive I.V. therapy. The shoe has a "D" ring attached to it with a Velcro-backed strap that can be detached from the arm strap to temporarily hang the fluids, the Flex-coil® Set and tubing in another area. The arm strap is a must if you want to improve hospital efficiency!!

STEP ONE

Insert your arm through the loop in the blue arm strap.



STEP TWO

Pass the strap behind your neck then let the attached white shoe hang downward.



STEP THREE

If necessary, readjust the position of the shoe utilizing the Velcro so the white shoe is as high as possible on the front of your shoulder. Then remove the swivel/coil from the cage and insert it in the shoe on the strap.



STEP FOUR

Attach the fluid bag (container) to one of the hooks on the white shoe.



STEP FIVE

Put a leash on the patient and you are ready to walk the patient while still receiving I.V. fluids. Cats can be carried.



STEP SIX

The white shoe with SAIV System and fluids can be disconnected from the arm strap at the Velcro connection and hung by the attached "D" ring.



INTERNATIONAL WIN, LIMITED

340 North Mill Road, Suite 6, Kennett Square, PA 19348-2853 USA • (610) 444-0170 • (800) 359-4946 • Fax (610) 444-0171
e-mail: info@internationalwin.com • www.internationalwin.com